English

5. SET RANGE-OF-MOTION

Depress extension ROM button and slide to desired setting. Depress flexion ROM button and slide to desired setting. Adjust opposite to the same settings of first hinge. NOTE: Settings automatically lock in place. Refer to standard hinge settings instruction sheet enclosed. Caution: The range-of-motion stops on both the medial and lateral hinges must be set the same.

6. LOCK HINGE

Lock hinge at a desired -10°, 0°, 10°, 20°, 30°, and 40° degrees. Bend leg to desired setting and engage lock on each hinge. This will lock the brace in place independent of ROM settings. Make sure ROM settings do not interfere with desired lock setting.

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INDICATIONS: Indicated for immediate application following injury or operative procedures to reduce pain and swelling and aid in healing.

CAUTION: Activity modification and proper rehabilitation are an essential part of a safe treatment program. If any additional pain or other symptoms occur while using this device, discontinue use and seek medical attention.

CAUTION: This device is not intended to prevent injury, or reduce or eliminate the risk of re-injury.

CAUTION: The range-of-motion stops on both the

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medial and lateral hinges must be set the same.

CAUTION: This device has been developed for the indications listed above, and may be damaged by unusual activities such as sports, falls, or other accidents, or exposure of the hinges to debris or liquids. To avoid the risk of accident and possible injury to the patient, discontinue use if such an incident has occurred.

Caution: Do not leave this device in a car on a hot day, or in any other place where the temperature may exceed 150F (65 C.) for any extended period of time.

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© 2015 Breg, Inc. All rights reserved.
MDSS GmbH